



1. Product Name

Cube Chair

2. Product Code

15929

3. Colour

Available in blue as standard. Other colours may be available on request.

4. Brief Description

Use as a recliner chair or fold up into a cube for storage/use as a climbing block.

5. Contents

- 1 x Cube Chair

6. Snoezelen® Stimulations

N/A

7. Best Use

Use on a stable floor.

8. Compatible ROMPA® Products

Many multisensory items can be used as passive or interactively whilst the client/s are sitting or using the Cube Chair.

If you wish to increase sensory stimulation for your client/s, please refer to our catalogue or website www.rompa.com for a great selection of items that provide a huge variety of physical, cognitive, olfactory, tactile and visual input.

9. Starting Up

- Unpack
- Sit on the cube chair
- Use to relax or to engage with activity

10. Detailed Description

The Cube Chair is designed to be moulded to fit body contours. Folds up easily into a cube which is convenient for storage. Can be used as a climbing block.

11. Safety

- Use indoors, in dry environments.
- Use on a flat, stable surface, capable of holding the product's weight and that of the person/people sat on it.



Please read all the following safety information before each use

ALWAYS SUPERVISE THE USE OF THIS PRODUCT

- Remove all packaging and either store this in a safe place out of reach of clients or dispose of this responsibly. Plastic bags and wrapping can be hazardous to children and vulnerable people.
- The Cube Chair contains a foam filling. Do not attempt to access the interior part of this product. Never allow children or vulnerable people to access the interior part of this product.

- Avoid sharp or pointed objects which could puncture the product. Take care with zips on clothing etc. Remove shoes.
- Do not place near sources of heat or fire that may damage the surface (radiators, fires, underfloor heating, hot central heating pipes etc.)
- Some of our products contain zips for manufacturing purposes only. Do not attempt to open these or to access the interior part of these products. Never allow children or vulnerable people to access the interior part of this product.
- Should any seams or the zip become damaged, contact ROMPA® immediately to arrange repair or replacement. Should the product require re-filling, contact ROMPA® immediately to discuss this.

12. Technical Specification

Size: 75 x 75 x 61cm
 Unfolded size: 119L x 63W x 76cmH

13. Installation

1. Unpack.
2. Place the furniture indoors, on a flat stable surface that is capable of holding the product's weight and that of the person sat on it.

14. Care and Maintenance

But please observe the following:-

- Always supervise the use of resources.
- Organise a regular cleaning schedule.
- Vinyl will crack, tear and rub off if subjected to sharp objects or continual rubbing by hard objects. Remove shoes, and avoid buckles, zips or sharp/hard/pointed objects coming into contact with the surface.
- Check seams regularly to ensure that these are not worn or split and the interior filling cannot be accessed.
- Wipe off accidental marks/liquids at the time they occur or are noticed.
- From time to time, if possible, lift floor cushion or move furniture to clean all surfaces and the floor underneath.
- Furniture and cushioning is intended to provide comfort and relaxation. Avoid eating and drinking whilst using these products.
- If the material for your furniture can be steam cleaned, please exercise great caution for the simple reason that when creating furniture, seams and stitching are necessary which create tiny holes where it is possible for liquid to get through into the inner foam. Should a lot of steam/water be accidentally used, there is potential for the inner foam to get saturated and be detrimental to your furniture.
- Do not place near sources of heat or fire that may damage the surface (radiators, fires, underfloor heating, hot central heating pipes etc.)

- Direct contact with bright sunlight may affect material over time.
- Do not saturate the material with lots of water.
- Some of our products contain zips for manufacturing purposes only. Do not attempt to open these or to access the interior part of these products. Never allow children or vulnerable people to access the interior part of this product.

PVC (Vinyl)

- Use a soft brush remove all loose dust/dirt
- Clean with a mild detergent (such as Lux soap flakes in luke-warm solution) applied to a sponge or damp cloth.
- This may be sanitised using a weak solution of Milton fluid.
- Do not use excessive amounts of water and always ensure the PVC is dry after cleaning.
- Do not use an abrasive cleaning pad.
- Do not use boiling/very hot water, washing up liquid / aggressive detergent or solvent based cleaner as this will cause the coating to discolour and go hard.
- Steam cleaning is not advised.

15. Troubleshooting

Not applicable

Further copies of these instructions can be downloaded at www.rompa.com

We hope you find our instructions invaluable. If you have any suggestions for improving them further your comments will be greatly received – please contact us at

producthelp@rompa.com