



## 1. Product Name

Tactile Totem by ROMPA®

## 2. Product Code

21010

## 3. Colour

As supplied. Colours may vary.

## 4. Brief Description

Tremendous touchy tower! Fabulous feely feature! Perfect palpable pillar! Lots to feel, discover and explore. Cushioned platform allows people to sit comfortably whilst exploring the textures.

## 5. Contents

- 1 x wood frame column with 12 interchangeable tactile boards
- 1 x cushioned platform
- 1 x top cushion of platform

## **6. Snoezelen® Stimulations**

- Sight
- Touch

## **7. Best Use**

Place in a suitable place so as clients may sit comfortably on any side of the cushioned platform.

## **8. Compatible ROMPA® Products**

Should you wish to change any of the Tactile Squares:  
20806-13, 20820-32 – Tactile Squares

## **9. Starting Up**

- Slide the wooden frame into the central space of the cushioned platform, marrying up the holes in the frame with the pre-drilled holes in the platform.
- Screw the frame to the cushioned platform.
- Replace any tactile squares that have been removed during transit.
- Secure the top cushion to the cushioned platform.
- Place the Tactile Totem in a suitable place.

## **10. Detailed Description**

Tremendous touchy tower! Fabulous feely feature! Perfect palpable pillar! Lots to feel, discover and explore. Cushioned platform allows people to sit comfortably whilst exploring the textures.

## **11. Safety**

- Always supervise the use of this product.
- This product is not intended to be climbed on or vigorously pulled or shaken.

## **12. Technical Specification**

Size: 110 x 110 x 30cm platform; 29.6 x 29.6cm squares

### 13. Installation

- Slide the wooden frame into the central space of the cushioned platform, marrying up the holes in the frame with the pre-drilled holes in the platform.





- Screw the frame to the cushioned platform.
- Replace any tactile squares that have been removed during transit. It is likely that two or more of the bottom squares have been removed for transit. Replace these once the platform and wooden frame are joined, but before the top cushion is attached.
- Secure the top cushion to the cushion platform using the Velcro.
- Place the Tactile Totem in a suitable place. Allow plenty of room for clients to sit comfortably on the Cushioned Platform.
- Remove any protective film off the boards with mirrors on.

## **14. Care and Maintenance**

1. Clean with a very slightly damp cloth and a mild cleaning product.
2. Avoid abrasive cleaning products and bleach.
3. The boards used are interchangeable – should you wish to attach them in another order, this is possible. Remove the screws securing the boards in each corner, and replace as required.

## **15. Troubleshooting**

*Not applicable*

Further copies of these instructions can be downloaded at [www.rompa.com](http://www.rompa.com)

We hope you find our instructions invaluable. If you have any suggestions for improving them further your comments will be greatly received – please contact us at [producthelp@rompa.com](mailto:producthelp@rompa.com)